Nord FASTE

EN	Teeth surface polishing paste
DE	Polierpaste für die Zahnoberflächen
BG	Паста за полиране на повърхности на зъбите
FR	Pâte à polir la surface des dents
EL	Πόστα γυαλισματος επιφάνειας δοντιών
IT	Pasta per lucidare la superficie dei denti
PL	Pasta do polerowania powierzchni zębów
ES	Pasta para profilaxis
TR	Diş yüzeyi parlatma macunu

INSTRUCTION FOR USE EN

DESCRIPTION

Nord FASTE is oil-free prophylaxis paste for tooth cleaning and polishing during professionally administered dental prophylaxis treatment.

COMPOSITION

Thickeners 30-50%, fillers 10-50%, solvent 5-15%, flavour 1-5%, excipients <3%, pigment <1%.

Nord FASTE does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

abrasiveness	coarse/medium/fine
--------------	--------------------

INTENDED PURPOSE AND CLINICAL BENEFITS

Nord FASTE improves aesthetic appearance; maintains dental function; protects biological structures.

CLINICAL INDICATIONS

- **To clean debris, plaque, calculus, stains, and other accretions from the teeth;**
- **For the teeth surface preparation (cleaning and polishing) prior to restorations and fluoride treatment;**
- **For recontouring, finishing and polishing the surface after restoration to restore normal contours, smooth margins, and to create a high luster shine, the overall appearance and longevity of the restoration itself.**

After evaluation of the teeth surface state and clinical situation dental professional may choose in one of several available levels of grit coarseness - fine (polishing), medium (polishing and cleaning) and coarse (cleaning).

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, Nord FASTE may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body - mouth. Tissues or body fluids contacted by the device - tooth, saliva, oral mucosa.

INTENDED USER

Nord FASTE is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

Nord FASTE is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

Nord FASTE is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of material is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No consumable components and accessories are supplied with the device.

INSTRUCTION FOR USE

1. Using suitable instrument take out a little of the paste and fill the prophylaxis cup.
2. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface.

3. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

WARNINGS

Do not use Nord FASTE for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. Nord FASTE does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device: If noticed any changes in the performance of the device, immediately remove applied material with suitable dental instrument. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center.

SHELF-LIFE

Shelf-life of Nord FASTE is 3 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Nord FASTE is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance will be introduced in EUDAMED as soon as it will start work.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF NRDFCD1	100x2g single dose cups coarse, peppermint
REF NRDFMD1	100x2g single dose cups medium, orange
REF NRDFFD1	100x2g single dose cups fine, apple

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