Nord SEAL

EN	Light curing pit and fissure sealant
DE	Lichthärtender Gruben- und Fissurenversiegler
BG	Лек втвърдяващ яма и улътнител за пукнатини
FR	Scellant photopolymérisable pour puits et fissures
EL	Ελαφρύ λάκκο σκληρυνσης και σφραγιστικό σχισμής
IT	Sigillante fotopolimerizzabile per fossette e fissure
PL	Światłoutwardzalny uszczelniacz do wgłębień i bruzd
ES	Sellador fotopolimerizable de fosas y fisuras
TR	İşikla sertleşen çukur ve fissür örtücü

INSTRUCTION FOR USE EN

DESCRIPTION

Nord SEAL is white, bioinert, opaque, light curing, highly flowable sealant especially suitable for sealing small pits and fissures.

COMPOSITION

Dental glass grinded 30-50%, methacrylate mixture 20-50%, silicon dioxide 1-5%, coinitiator <1%, photoinitiator <1%, inhibitor <1%, stabilizer <1%, opacifier <1%. Nord SEAL does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

light curing	20 s
depth of cure	3.79±0.07 mm

INTENDED PURPOSE AND CLINICAL BENEFITS

Nord SEAL improves aesthetic appearance; maintains dental function; protects biological structures.

CLINICAL INDICATIONS

- For sealing small pits and fissures.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

Nord SEAL should not be used with products containing eugenol as eugenol may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, Nord SEAL may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth, saliva.

INTENDED USER

Nord SEAL is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental sealants. There is no need for specific training.

STERILITY

Nord SEAL is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

Nord SEAL is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of sealant is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as application tips, are supplied with the device.

INSTRUCTION FOR USE

1. Prepare surface as always. Cleaning with oil-free prophylaxis paste, such as Nord FASTE or pumice is recommended.
2. Apply layer of etch, such as Nord GEL on all surface to be etched. Leave etch in place for 30 seconds. Rinse with water and dry with air.
3. The working area must be kept dry and not contaminated.
4. Apply a thin layer of Nord SEAL immediately onto etched surface: twist a disposable dispensing tip onto the syringe. Lightly pressing the plunger of the syringe, apply sealant onto mixing pad. Then apply onto surface using disposable micro applicator.
5. Light curing for 20 seconds. Use LED polymerization lamp with light intensity 1200mW/cm² in full mode, not ramp or pulse mode. Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use.
6. Inspect sealant for complete coverage and voids.

7. Wipe surface with cotton roll and check for hardness and retention.

WARNINGS

Sealant is highly flowable do not force syringe plunger. After the desired amount of material extruded, immediately remove application tip and close the syringe cap, so that the material is not unlighted. The material is sensitive to light. Avoid too long manipulation time under intensive lighting. Do not use Nord SEAL for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. Nord SEAL does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If noticed any changes in the performance of the device, immediately remove applied material with suitable dental instrument. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center.

SHELF-LIFE

Shelf-life of Nord SEAL is 4 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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